

COLEMAN KNITTING MILLS PRODUCT INFORMATION FOR POLYESTER MASK

MATERIAL

Coleman masks are made with an 8.4 ounce tight weave polyester. Polyester is a synthetic fiber that wears well and can repeatedly be washed in accordance with fabric instructions.

WASHING INSTRUCTIONS

Polyester can be hand cleaned or by a standard washing machine. Machine washing should typically be done with warm or cool water on a normal setting. The detergent utilized should be compatible with general polyester cleaning as described on the manufacturer's label.

Air dry or tumble dry at medium temperature.

VISUAL INSPECTION OF SEAMS AND STITCHING

Coleman recommends that between each washing and use that a visual inspection of the seams and stitching. If there is any area that is coming apart, thread deterioration or any other issue that differs from the remainder of the garment that use be immediately discontinued.

VISUAL INSPECTION OF FABRIC

Coleman recommends that between each washing and use that a visual inspection of the fabric for fraying, holes, cuts or other deteriorations that may compromise the barrier purpose of the mask. If there is any area located on the mask that contain the foregoing the mask's use should immediately be discontinued.

LIFE OF PRODUCT

Coleman's testing of the polyester mask indicates that it may be washed in excess of 100 times without any significant degradation in the: a) material; and b) stitching. Regardless, batches of fabric may vary and a visual inspection is required for continual use.

LIMITED WARRANTY

Coleman warrants the mask and will repair or replace the same as follows: 1) the mask was used for the intended purpose of providing a limited barrier for a single individual; 2) the mask defect is related to stitching, hemming, surging, or other such similar manufacturer defect; 3) the mask was washed pursuant to the instructions provided and with a detergent compatible with polyester; and 4) the customer is the original purchaser of the mask from Coleman. Coleman issues no other implied or express warranty. Coleman makes no claims that the mask has received an FDA approval for a Level II, Level III or Level IV medical gown.

The customer shall contact Coleman for redeeming any warranty by calling (801) 621-2072 for further instructions.

COLEMAN KNITTING MILLS PRODUCT INFORMATION FOR MICROSUEDE GOWN

MATERIAL

Coleman gowns are made with a tight weave microsuede. Microsuede is a polyester microfiber with a water-repellent finished surface resembling that of suede. It is a synthetic fiber that wears well and can repeatedly be washed in accordance with fabric instructions.

WASHING INSTRUCTIONS

Microsuede can be hand cleaned or by a standard washing machine. Machine washing should typically be done with cold water on the delicate or hand-wash setting. The detergent utilized should be compatible with general microsuede cleaning as described on the manufacturer's label. It is preferable that microsuede be washed separate from other fabrics because it tends to collect lint from other materials during the washing/drying.

Air dry or tumble dry at medium to low temperature (not to exceed 140 degrees if at all possible).

VISUAL INSPECTION OF SEAMS AND STITCHING

Coleman recommends that between each washing and use that a visual inspection of the seams and stitching. If there is any area that is coming apart, thread deterioration or any other issue that differs from the remainder of the garment that use be immediately discontinued.

VISUAL INSPECTION OF FABRIC

Coleman recommends that between each washing and use that a visual inspection of the fabric for fraying, holes, cuts or other deteriorations that may compromise the barrier purpose of the gown. If there is any area located on the garment that contain the foregoing the gown's use should immediately be discontinued.

LIFE OF PRODUCT

Coleman's testing of the microsuede gown indicates that it may be washed in excess of 100 times without any significant degradation in the: a) material; and b) stitching. Regardless, batches of fabric may vary and a visual inspection is required for continual use.

LIMITED WARRANTY

Coleman warrants the gown and will repair or replace the same as follows: 1) the gown was used for the intended purpose of providing a limited barrier at a medical/nursing facility; 2) the gown defect is related to stitching, hemming, surging, or other such similar manufacturer defect; 3) the gown was washed pursuant to the instructions provided and with a detergent compatible with microsuede; and 4) the customer is the original purchaser of the gown from Coleman. Coleman issues no other implied or express warranty. Coleman makes no claims that the gown has received an FDA approval for a Level II, Level III or Level IV medical gown.

The customer shall contact Coleman for redeeming any warranty by calling (801) 621-2072 for further instructions.